

# Geratherm®

Medical Warming Systems

www.geratherm.com



Warmth that saves lives



RescueWarmingSystem

UniqueResc+

**UniqueResc<sup>+</sup>** – the most effective form of mobile hypothermia prevention in the rescue field. Its unique microchip technology in combination with its light-weight heatconducting fabric is responsible for the revolutionary characteristics of this elegant electrical RescueWarmingSystem.

### PATIENT | Benefits

- **Rapid stabilisation of the patient**  
patient calms down and is able to communicate much more quickly
- **Prevents shivering due to cold**  
by active and even conduction of warmth
- **Reduction of fear**  
patient feels much more comfortable - even during transportation



Medium blanket:  
50 x 96 cm

Small blanket: 48 x 49 cm

### USER | Benefits

- **Small, light-weight and foldable**  
fits into emergency backpacks
- **Plug and warm**  
by integrated control unit
- **Versatile and mobile**  
by battery or on-board power supply



// State-of-the-art quality management and strict adherence to all standards ensure the safety and longevity of our products.

# The RescueWarmingSystem

Mobile. Light-weight. Universal.

**UniqueResc+** – the latest polymer technology combined with a special surface coating create an ideal symbiosis between the very highest safety standards and maximum efficiency in our warming blankets.

## EXAMPLES OF APPLICATION

### → Patient in sitting position

Application of UniqueResc+ small (1) or medium (2)

Example: Rescue of road traffic accident victims



### → Patient in lying position

Application of UniqueResc+ small (3) or medium (4)

Example: Transport from accident site to hospital, transportation of patients

(5) Application in the special Geratherm Thermobag



## ACCESSORIES (extra charges apply)

Geratherm special battery (Li-Ion) with charging unit and fastening belt

Transport bag for UniqueResc+ (with space for blanket and battery)

Individual on-board power supply plugs



„While I was being rescued, the warmth from the UniqueResc+ gave me a feeling of security and safety, and so I was not afraid.“ Daniela M. (27), Munich

## Technical Facts

	Geratherm UniqueResc+	
Blanket type	medium	small
Article number	8001110611	8 001110612
Designation	UniqueResc+ AW-R 130	UniqueResc+ AW-R 150
Design	actively warming blanket with integrated control unit	
Surface	special PU film, blood and fluid resistant	
Cleaning	wipeable, resistant to disinfectant	
Weight in kg approx.	0.9	0.5
Dimensions in cm approx.	50 x 96	48 x 49
Control unit	integrated into the blanket	
Temperature	temperature pre-selection by touch 37.0 °C / 40.5 °C	final temperature pre-set by manufacturer at 40.5 °C
Temperature display	via LED (intermittent when final temperature is reached)	
Fault display	acoustic and visual alarm	
Safety components	auto-check, temperature measurement between warming system and patient, permanent temperature monitoring, emergency shut-down in the event of impermissible temperatures, shut-down in the event of excess current or interruption of power supply	
Plug used	universal motor vehicle plug	
Energy supply	on-board voltage (12 - 24 Volt DC) or Geratherm special battery (6 A at 12V or 12A at 24V)	
Operating temperature	-30 °C to + 40 °C	
Storage temperature	-35 °C to + 70 °C	

Subject to technical modifications | cover picture: © copyright STOCK4B-RF

If you have any questions or require further information about our Geratherm Patient Warming Systems please do not hesitate to contact us.



### GERATHERM MEDICAL AG - Germany

phone: +49 36205 98-0 • fax: +49 36205 98-116 • info@geratherm.com • www.geratherm.com